normo mar[®] spray

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Aloe Vera y Eucalipto SPRAY NASAL DESCONGESTIONANTE

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Aloe Vera and Eucalyptus

DECONGESTANT NASAL SPRAY

Normomar[®] Sea Water Hypertonic with Aloe Vera and Eucalyptus – Nasal Spray

Nasal decongestant designed to aid during pathological processes occurring with secretions. Its daily use helps clearing, protecting and moisturizing the nasal passages. The hypertonic sea water solution has a strong moisturizing effect, boosted with Aloe vera that helps and prolongs the moisturizing power of the sea water, which helps avoiding an excessive microbiological proliferation found within the nasal passages. Eucalyptus provides a pleasant and fresh aroma.

Composition:

Sea water (sodium chloride 2.2%), purified water, propylene glycol, ethoxylated hydrogenated castor oil, aloe vera, eucalyptus flavor, mix of natural tocopherols and trisodium citrate dihydrate.

Recommended dose:

For adults and children from 6 years of age: Apply 2 to 6 sprays in each nostril per day, or as often as necessary.

Instructions for use:

1. Remove the protective cap.
2. Introduce gently the nasal applicator in one nostril and press the pump to apply the product.
3. Let drain the excess solution in order to extract the mucus, and then blow.
4. Repeat the process with the other nostril.
5. Clean the tip with water and dry.
6. Cover the nasal applicator with the protective cap until the next use.

Contraindications and adverse effects:

• No adverse effects have been described at the recommended therapeutic doses, but if you experience side effects, consult your usual doctor or pharmacist.

• Please report any adverse events related to this product to the manufacturer, distributor or the national authority in your country.

Warnings and precautions:

- For hygienic reasons and to prevent the transmission of pathogens, this container should not be used by more than one person.
- In case of a known or suspected allergy to any of the components of this medical device, it is advisable not to use.
- Keep out of reach of children.

Conservation and expiry date:

- Do not use the product after the expiry date shown on the box and the bottle.
- Maintain in a dry and cool place. Maintain below 40°C and protected from direct sunlight or other heat sources.
- Deposit the bottle and the protective cap in the corresponding containers.

Presentation:

Pressurized spray bottle of 120 ml with nasal applicator in medium force.

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LABORATORIOS NORMON, S.A.